

<u>PRECAUTIONARY STATEMENTS</u>	
Hazards to Humans and Domestic Animals	
DANGER. Corrosive. Causes irreversible eye damage. Harmful if swallowed. Do not get in eyes or on clothing. Wash thoroughly with soap and water after handling and before eating, drinking, chewing gum, or using tobacco. Remove contaminated clothing before reuse.	
<u>FIRST AID</u>	
If in Eyes:	Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lenses, if present, then continue rinsing eye. Call a poison control center or doctor for treatment advice.
If Swallowed:	Call poison control center or doctor for treatment advice. Have person sip a glass of water if able to swallow. Do not induce vomiting unless told to do so by the poison control center or doctor. Do not give anything by mouth to an unconscious person.
Have the product container or label with you when calling poison control center or doctor, or going for treatment.	
<u>NOTE TO PHYSICIAN</u>	
Probable mucosal damage may contraindicate the use of gastric lavage.	
<u>ENVIRONMENTAL HAZARDS</u>	
This product is toxic to fish and other aquatic organisms.	
<u>PHYSICAL/CHEMICAL HAZARDS</u>	
Keep away from combustible, organic or other readily oxidizable materials. Do not mix with any other chemicals.	



**VistaTab™
Dental Waterline Cleaner Tablets**

V9200

**For the Cleaning and Control of
Microbial Contaminants in
Dental Unit Waterlines**

Active Ingredients:
Sodium Chlorite 4.0%
Sodium Dichloroisocyanurate dihydrate . 1.0%
Other Ingredients: 95.0%
Total: 100.0%

EPA Reg. No. 70060-25-73593
EPA Est. No. 73593-OH-001

** Sodium chlorite in this product is from Spain*

KEEP OUT OF REACH OF CHILDREN

DANGER

This box contains: 15 – 1.5 gram tablets
Net Weight 22.5 grams

Vista Research Group, LLC
P.O. Box 321, Ashland, OH 44805-0321

DIRECTIONS FOR USE

It is a violation of Federal law to use this product in a manner inconsistent with its labeling.

INITIAL TREATMENT:

1. Fill the 750 ml dental unit water bottle with potable water and add two (2) 1.5 gram *VistaTab* Dental Waterline Treatment Tablets. Allow the tablets to dissolve.
2. Flow about half of the treated water through the dental waterlines and let the treated water sit within the lines for 5 minutes. Purge the remaining water through the lines.
3. Repeat steps 1 and 2.
4. Rinse and refill the dental unit water bottle with potable water and flush at least 500 ml of water through the lines to remove any residual chemical.

ROUTINE TREATMENT:

1. Every 1 to 4 weeks, fill the 750 ml dental unit water bottle with potable water and add one (1) 1.5 gram *VistaTab* Dental Waterline Treatment Tablet. Allow the tablet to dissolve.
2. Flow about half of the treated water through the dental waterlines and let the treated water sit within the lines for 5 minutes. Purge the remaining water through the lines.
3. Rinse and refill the dental unit water bottle with potable water and flush at least 500 ml of water through the lines to remove any residual chemical.

Note: Periodic cleaning may be done on any day of the week.

STORAGE AND DISPOSAL

Do not contaminate water, food, or feed by storage and disposal. Store in a cool, dry area away from sunlight and heat and in an area inaccessible to children. Keep product from contacting air prior to use. Place empty foil pouch in trash.